- Avoid exposing the wound site to direct sunlight and do not use tanning beds or lamps while the adhesive film is intact.
- Showers or light baths are permitted. However, do not rub, soak, or expose the wound site to prolonged moisture (including swimming) until the film has naturally come off (usually in 5 to 10 days) and your doctor has determined that the wound has healed properly.
- Do not apply liquid or ointment medications to wounds closed with BlauGlue® Topical Skin Adhesive. These substances can weaken the polymerized film, leading to separation and/or dehiscence of the wound edge.
- Inform your doctor about any discomfort or other concerns related to your wound.

Sterilization

BlauGlue® Topical Skin Adhesive is terminally sterilized by gamma radiation. Do not re-sterilize. Do not use if the packaging is open or damaged. Discard any unused or opened materials after completing medical treatment.

Elimination

- The adhesive will naturally disappear from the skin in about 3 days and does not need to be removed.
- Excess BlauGlue® can be removed before it dries with a clean cloth. Once dry, unwanted BlauGlue® can be softened with petroleum jelly and removed.

Warnings and precautions

- The topical skin adhesive product BlauGlue® is composed of a fast-setting adhesive capable of adhering to most body tissues and many other materials, such as surgical gloves and stainless steel. Unintentional contact with any body tissue and any surface or equipment that is not disposable or cannot be easily cleaned with a solvent such as acetone should be avoided.
- Polymerization of the topical skin adhesive BlauGlue® can be accelerated in the presence of water or liquids containing alcohol. BlauGlue® topical skin adhesive should not be applied to wet wounds.
- BlauGlue® topical skin adhesive will not adhere to skin previously covered with petroleum jelly. Therefore, avoid using petroleum jelly in any area of the skin where BlauGlue® topical skin adhesive will be used.
- BlauGlue® topical skin adhesive should not be applied to the eye. If contact occurs with the eye, rinse thoroughly with saline solution or water. If adhesive residue remains, apply an appropriate topical ophthalmic ointment to help loosen the bond and contact an ophthalmologist.
- When closing facial wounds near the eye with BlauGlue® topical skin adhesive, position the patient so that the flow of adhesive does not reach the eye. The eye should be closed and protected with gauze. Placement of petroleum jelly near the eye may be effective in preventing inadvertent flow of adhesive into the eye. The use of liquid adhesives near the eye may cause the eyelids of some patients to close inadvertently. In some of these cases, general anesthesia and surgical intervention may be needed to open the eyelid.
- BlauGlue® topical skin adhesive should not be used internally or under the skin because the tissue does not absorb the polymerized material and can cause a foreign body reaction.
- Avoid excessive pressure from the applicator tip against the wound area or surrounding skin. This can force the wound edges and allow the adhesive to seep into the wound. BlauGlue® topical skin adhesive should be applied with a gentle brushing motion of the applicator tip over the approximate wound edges.
- BlauGlue® topical skin adhesive should not be used on areas of high skin tension, such as elbows, knees, and knuckles, without immobilization of the joints or additional support from a device (such as sutures) to relieve tension.
- Wounds treated with BlauGlue® topical skin adhesive should be monitored for signs of infection. Wounds with signs of infection, such as swelling, redness, heat, pain, and purulent drainage, should be evaluated and treated according to standard practice for infections.
- BlauGlue® topical skin adhesive should not be used on areas of wounds that will be subjected to repeated or prolonged moisture or friction.
- BlauGlue® topical skin adhesive should only be used after wounds have been thoroughly cleaned, debrided, and treated according to standard surgical practices. Local anesthesia may be used when necessary to ensure sufficient cleaning and debridement.
- Polymerization of BlauGlue® topical skin adhesive generates a small amount of heat and should not be applied to sensitive tissues that may be affected by such heat. Applying BlauGlue® in a thin, continuous layer over a dry wound and allowing time for polymerization will minimize the sensation of heat.
- The adhesive should not be applied to wet wounds or wounds that have excess moisture. Excess moisture can accelerate polymerization, generating excess heat.
- BlauGlue® topical skin adhesive is packaged for use on a single patient. Discard any unused material after each wound closure procedure.
- Do not re-sterilize BlauGlue® topical skin adhesive.
- Do not place BlauGlue® topical skin adhesive in a package, tray, or kit that will be sterilized before use. Exposure of sterilized BlauGlue® topical skin adhesive to excessive heat (such as in autoclaves or ethylene oxide sterilization) or radiation (such as gamma or electron beam) will increase its viscosity and may render the product unusable.

Other information

- Store in a cool, dry place. Avoid high humidity and excessive heat.
- For shelf life, refer to the expiration date on the packaging.
- Dispose of in accordance with local regulations.

Warning: The sale of this product is restricted to a healthcare professional or by order of a physician (or duly authorized healthcare professional). Any serious incident related to the product should be reported to the manufacturer and competent authority.

Fabricado por:
Synkotech Biocompatible Materials S.L.
Av. Països Catalans, 18
43007 Tarragona (Spain)
<https://synkotechbio.com>

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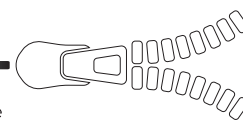
BlauGlue®

Topical skin adhesive

Tissue adhesive for the surgical incisions and lacerations closure

FOR EXTERNAL USE ONLY

Description

BlauGlue® Liquid Skin Adhesive is a medical device intended for topical use to close skin wounds. It is a fast-polymerizing, non-cytotoxic liquid based on methylidene malonate based.

BlauGlue® is applied to the skin as a viscous liquid that polymerizes to bind the approximated wound edges in minutes. In vitro studies have shown that BlauGlue® acts as a bacterial barrier as long as the adhesive film remains intact.

Indications for use

BlauGlue® Liquid Skin Adhesive is designed for topical use only, to hold together the skin edges of incisions and lacerations that are under minimal tension and easily approximated. Where significant tension exists on incisions or lacerations, BlauGlue® topical skin adhesive should be used in conjunction with deep dermal stitches.

Contraindications

- Do not use on wounds with evidence of microbial, bacterial or fungal infection, gangrene, or decubitus etiology.
- Do not use on mucous surfaces or through muco-cutaneous junctions (e.g., oral cavity, lips), or on skin that may be regularly exposed to bodily fluids or with dense natural hair (e.g., scalp).
- Do not apply internally, inject intravascularly, or ingest.
- Do not use on patients with known hypersensitivity to adhesives.

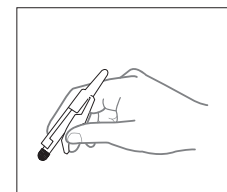
Instructions (Please use under the supervision of a healthcare professional)

PREPARE THE APPLICATION SITE:

Clean and dry the skin thoroughly before applying. Make sure to remove all traces of moisturizers, lotions, lotion soaps, or lanolin from the application site.

Application:

- The wound site must be thoroughly cleansed before using BlauGlue® topical skin adhesive. Standard surgical practices for wound preparation before product application, including cleaning, irrigation, debridement, and achieving hemostasis, should be used. There should be low tension at the wound edges. Additional support to the wound can be provided by using deep dermal sutures.
- Dry the wound area with a sterile gauze. Moisture can accelerate the polymerization of BlauGlue® topical skin adhesive and may affect wound closure results.
- To prevent inadvertent flow of liquid BlauGlue® topical skin adhesive to undesired areas of the body, the patient's wound area should be kept in a horizontal position, and the adhesive should be applied from above the wound.
- BlauGlue® topical skin adhesive should be used immediately after crushing the inner blister because the adhesive will polymerize in the applicator, rendering the device useless.
- Remove the applicator from the aluminum pouch.
- Hold the applicator with the tip facing upward.
- Apply pressure to the applicator to break the inner blister.
- Invert and gently squeeze the applicator to extract the liquid adhesive.
- If necessary, approximate the wound edges using sterile gloves or forceps.
- Slowly apply the BlauGlue® topical skin adhesive liquid in a thin, continuous layer to the surface of the approximated wound edges with a gentle brushing motion.
- The adhesive should cover a target area of approximately 0.5 cm on each side of the wound.
- Manual tissue approximation should be maintained for two minutes, especially if the wound appears to be under tension. The adhesive may remain "sticky" for up to four minutes.
- Complete polymerization is expected when the BlauGlue® topical skin adhesive layer is no longer sticky.
- Do not apply liquid or ointment medications to wounds closed with BlauGlue® topical skin adhesive. These substances can weaken the polymerized film, leading to wound edge separation and/or dehiscence.
- Dry protective dressings, such as gauze, can only be applied after the adhesive film is completely polymerized and is not tacky to the touch.
- If a dressing or bandage is applied before polymerization is complete, the dressing or bandage may adhere to the film. Then, the film may detach from the skin when the dressing is removed, possibly causing wound dehiscence.

Additional information

- No additional or special care is needed for wounds closed with BlauGlue® Topical Skin Adhesive. However, the following information should be shared with the patient as needed. A dry protective dressing should be applied to children or other patients who cannot follow these instructions for proper wound care.
- Do not pull or touch the wound or the polymerized film of BlauGlue® Topical Skin Adhesive. Touching the film can disrupt adhesion to the skin and cause wound dehiscence.