BlauProtect®

SPRAY / SPONGE / WIPE

Skin Barrier Film for Skin Protection

FOR EXTERNAL USE ONLY

Description

BlauProtect® is a fast-drying, non-cytotoxic skin barrier film based on malonate polymer. BlauProtect® forms a transparent, flexible, and breathable film.

Indications for use

BlauProtect® is designed to protect intact or irritated skin from moisture, urine, feces, digestive juices, sweat, and wound dressings. It also protects against friction and shearing caused by clothing, shoes, or any materials that may damage the skin. Additionally, it protects the skin from irritation that can be caused by adhesive products.

Contraindications

Do not apply near or around the eyes or on mucous membranes. Do not apply if allergic to any of its components. Do not apply directly on deep, open, or bleeding wounds. Do not apply on chronic wounds (persistent or non-healing wounds). Do not apply on infected areas.

Instructions for use

- BlauProtect® Skin Barrier Film is easy to apply in any of its formats and does not require special equipment or knowledge.
- Clean and thoroughly dry the skin before applying. Make sure to remove all traces of moisturizers, lotions, lotion soaps, or waxes from the application area.
- When applying:
 - For BlauProtect® Spray Skin Barrier Film, apply by holding the sprayer 8-12 cm away from the skin, pressing the sprayer to create a protective film on the affected area.
 - For BlauProtect® Sponge and BlauProtect® Wipe Skin Barrier Film, dispense a sufficient amount with the sponge or wipe to create a protective film on the affected area.
- Allow the product to dry for at least 30 seconds.

Additional Information

- If applying BlauProtect® Skin Barrier Film in a skin fold or another skin-to-skin contact area, separate the folds and allow the product to dry completely before returning the skin to its original position.
- Ensure that BlauProtect® Skin Barrier Film has fully dried before covering it with clothing, adhesive products, dressings, or incontinence products.
- After application, you can continue your cleansing routine; the product will remain intact.
- If additional product application is required after the first application, ensure that the first application is completely dry.
- Disposal.
- The protective film will naturally disappear from the skin and does not need to be removed.

Warnings & Precautions

- If BlauProtect® comes into contact with the eye, rinse immediately with water. Do not rub or attempt to separate the eyelids. Seek medical attention.
- If skin symptoms persist or worsen, seek medical attention.
- Avoid applying on wet or moist skin.
- Do not use under medication patches.
- The use of other barrier products, ointments, creams, or lotions may reduce the product's effectiveness.
- Keep out of reach of children.
- The product in its fluid form is flammable; use it in well-ventilated areas. Avoid use near flames and ignition sources.

Other Information

Store in a dry place. Avoid excessive heat.
For shelf life, refer to the expiration date on the packaging.
Dispose of according to local regulations.

| Ref. | Presentation | Presentation / Box | Box / Secondary box |
|-------------|------------------|--------------------|---------------------|
| BP_Spray25 | Spray de 25 mL | 1 | 50 |
| BP_Spray100 | Spray de 100 mL | 1 | 10 |
| BP_Spray200 | Spray de 200 mL | 1 | 10 |
| BP_Sponge1 | Esponja de 1 mL | 5 | 10 |
| BP_Sponge3 | Esponja de 3 mL | 5 | 10 |
| BP_Wipe1 | Toallita de 1 mL | 30 | 10 |

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